
Adult & Adolescent Water Rescue Manikins



Item Numbers
1326 & 1327



 **Simulaids**

Features and Construction

The Adult Water Rescue manikin's skeleton consists of coated steel cable, fittings and hardware. The manikin is articulated, and can be used in various water lifesaving exercises including top and bottom rescue.

Adult

- Manikin weight- approximately 40 lbs.
- Manikin submerged weight – approximately 105lbs.
- Can be additionally weighted to reach a submerged weight of 140lbs.
- Cables and limbs are replaceable (Contact our customer service department).

Adolescent

- Manikin weight- approximately 20 lbs.
- Manikin submerged weight – approximately 50lbs.
- Cables and limbs are replaceable (Contact our customer service department).

Basic Use:

Top Water Rescue:

The adult Water Rescue manikin is designed for water safety programs. The manikin takes on weight by filling with water through the small holes in the legs and arms. The manikin's torso may be filled with water by opening the valve located on the shoulder. When the manikin is filled it will sink to around mouth level.

Bottom Water Rescue:

To fully submerge the Adult Water Rescue manikin, remove the vinyl cover and plastic insert located in the abdomen of the manikin. Add about 10 lbs. of weight (i.e., a brick) Replace the plastic insert and vinyl cover. The manikin will now fully submerge.

Important: We strongly recommended that a flotation device and/or tether be affixed to the manikin in those situations where loss is possible.

STORAGE: Drain the water rescue manikin after each use. Do not turn the manikin upside down to drain. Repositioning of the extremities will allow the water to drain out of the holes in the posterior aspects of the arms and legs while the manikin is lying on its back. To remove water from the chest cavity, open the shoulder valve minimally and position the manikin on a slant to facilitate drainage. Store the manikin at room temperature. In cold environments the manikin will harden and will return to normal flexibility when returned to a warmer temperature.

RETURN POLICY: Should it be necessary to return an item for credit for any reason, contact our Customer Service Department to obtain an RGA Number. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions or wish further information on any product we manufacture call or write our Customer Service Department.

LIMITED WARRANTY: Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required above.



 **Simulaids**



Manufacturers of Training Manikins, Casualty Simulation Kits, Medical Training
Devices

PO BOX 1289 - 16 Simulaids Ave Saugerties NY 12477

(845)-679-2475

Toll Free: (800)-431-4310 Fax: (845)-679-8996

www.simulaids.com