



Assembly & Operation Instructions

PEDIATRIC WATER RESCUE MANIKIN



These manikins are intended to be used as an instructional aide in the teaching and demonstration of Pediatric Water Rescue.



Assembly & Operation Instructions

PEDIATRIC WATER RESCUE MANIKIN




Thank you for purchasing a pediatric water rescue manikin. The product comes with two tools; a faucet connection hose and an air inflation bulb.

To simulate realistic weight and to use in water rescue and safety training, fill manikin with water using the following instructions.

Fill Manikin

- Steps:**
1. Hold manikin upside down or lay in sink.
 2. Open both valves on bottom of manikin's feet. Rescue Timmy has one metal valve on the left foot and one aqua drain valve with an internal flap on the right foot.

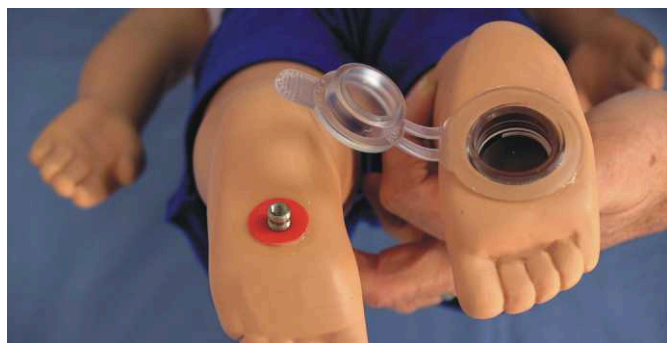

3. Attach hose from faucet to left metal valve on baby's foot or insert through internal flap of aqua valve on Timmy's foot.
4. Fill manikin with warm water until water begins to flow from the opened metal valve on right foot.
5. Shut off flow of water.
6. Close valve on right foot.
7. Remove hose from valve on left foot.
8. Gently squeeze air from manikin until water flows from left valve. Close valve tightly.
9. Let manikin lay 1 or 2 minutes with head down.
10. Repeat steps 2 through 8 making sure all air is out of the manikin.

NOTE: Do not store manikins filled with water. Microbiologic growth could occur and extreme temperature will effect the manikin.

Drain Manikin

After every use the manikin must be drained of all water and filled with air before storing it.


- Steps:**
1. Place manikins' feet into sink and open both valves. Insert pencil or dowel (DO NOT USE PEN) through internal flap of aqua drain valve for Timmy manikins only.
 2. Let drain 30 minutes.





Assembly & Operation Instructions

PEDIATRIC WATER RESCUE MANIKIN

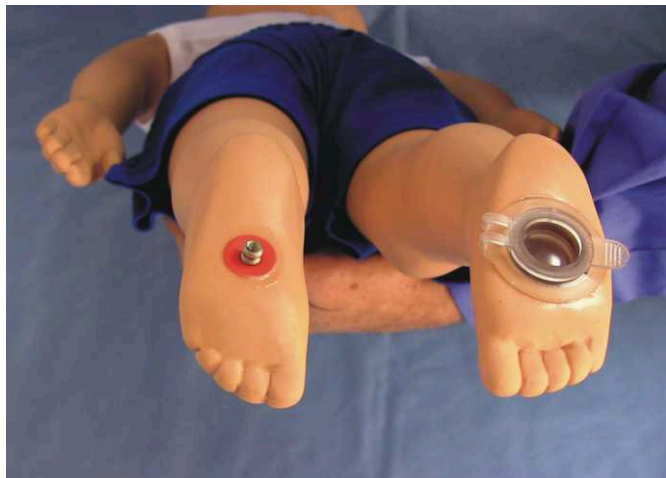

3. Holding manikin over sink squeeze excess water from manikin.
4. Repeat steps 2 and 3.
5. Close metal valve on left foot of baby manikin. Close aqua drain valve cap on Timmy manikin.
6. Attach tubing from inflation bulb to metal valve on right foot. Inflate manikin with air by pumping bulb. **DO NOT OVERINFLATE.** Grasp valve with tubing and turn to close.
7. Remove inflation bulb assembly.

Products

1350 Rescue Cathy - Newborn Size - Water filled 7 lbs.
Sh. Wt.: 5 lbs.; Size: 26" x 8" x 8"

1352 Rescue Billy - 6 to 9 Month Old - Water filled 12 lbs.
Sh. Wt.: 7 lbs.; Size: 26" x 8" x 8"

2252 Carry Bag for Rescue Cathy and Billy

1351 Rescue Timmy - 3 Year Old - Water filled 20 lbs.
Sh. Wt.: 10 lbs.; Size: 34" x 11" x 7"

2260 Carry Bag for Rescue Timmy

Warranty

Simulaids warrants their products to be free from defects in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date on the invoice of the product shipment to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. This warranty does not cover moulage products. Only products manufactured at the Simulaids plant in Saugerties, NY receive this limited warranty status. All other products sold through Simulaids, but manufactured elsewhere, are subject to the warranties supplied by the product manufacturer. These warranties may differ from the Simulaids' warranty.

Return Policy

Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory.

At no time will any product be accepted without proper return authorization issued by Simulaids. Please contact our Customer Service Department to arrange a return. Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Serial number and invoice number from the agency through whom the product was purchased must be provided for warranty repairs. No return authorization number will be provided without this information.

Should you have any questions or wish further information on any product we manufacture call or write:

Customer Service Department

PO Box 1289 – 16 Simulaids Drive
Saugerties, NY 12477

800-431-4310 or 845-679-2475

www.simulaids.com

info@simulaids.com



Manufacturers of Training Manikins, Casualty Simulation Kits & Medical Training Devices

PO Box 1289 • 16 Simulaids Drive • Saugerties, NY 12477
(845) 679-2475 • 1-800-431-4310 • FAX: (845) 679-8996
www.simulaids.com