*Assembly & Operation Instructions*

# **SANI-MAN, SANI-BABY & SANI-CHILD**

**The purpose of this instructional is to familiarize the user with the function, care and maintenance of the Simulaids' Sani-Man, Sani-Child & Sani-Baby training manikins.**

## Assembly & Operation Instructions

# SANI-MAN, SANI-BABY & SANI-CHILD

**Thank you for buying an American made product.** The purpose of this instructional is to familiarize the user with the function, care and maintenance of the Simulaids' Sani-Baby training manikin. It is not meant to provide instructions in CPR or other emergency life procedures. Please read it thoroughly. All Simulaids manikins are designed and manufactured to exacting standards and will provide years of effective and trouble-free use when the simple steps of care and maintenance are regularly performed.

**Note:** The manikin you have just purchased is a member of Simulaids' Sani-Man line. These manikins serve many individuals during a training session. In order to eliminate cross contamination, or the possibility of cross-contamination, these manikins are designed with a "single use airway." That means that only one person uses the plastic bag airway system.

When the individual completes his/her airway inflations, the bag is to be removed and the next student will place his/her airway in the manikin. The airway can be retained by an individual for further use during that training session, but the airway should be discarded at the end of that session. This is the only recommended procedure for eliminating cross-contamination.

### Airway Removal

- Steps:**
1. Remove chest overlay from shoulders and peel down.
  2. Remove the face shield bag by pulling it out of the mouth.

### Airway Installation

- Steps:**
1. Fold opened end of face shield bag as shown below. Tilt head back and place folded end up through the mouth.
  2. With head tilted back lay the lung flat against the chest and pull the overlay back up into place and snap.
  3. Form the face shield to cover face from nose to chin.

### Warranty

Simulaids warrants their products to be free from defects in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date on the invoice of the product shipment to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. This warranty does not cover mouldage products. Only products manufactured at the Simulaids plant in Saugerties, NY receive this limited warranty status. All other products sold through Simulaids, but manufactured elsewhere, are subject to the warranties supplied by the product manufacturer. These warranties may differ from the Simulaids' warranty. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory.

### Return Policy

At no time will any product be accepted without proper return authorization issued by Simulaids. Please contact our Customer Service Department to arrange a return and obtain a RGA number. Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Serial number and invoice number from the agency through whom the product was purchased must be provided for warranty repairs. No return authorization number will be provided without this information.

Should you have any questions or wish further information on any product we manufacture call or write our Customer Service Department.

Part #	Description
2121	Sani-Baby
2124	Sani-Baby 4-Pack
2125	Sani-Baby Face Shield Lung System (100)
2252	Baby Carry Bag
2260	Baby Carry Bag 4-pack
2140	Sani-Child
2131	Sani-Man

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