



*Assembly & Operation Instructions*

# **ECONOMY ADULT SANI-MANIKIN**



**This manikin is intended to be used as an instructional aide in the teaching and demonstrating of CPR procedures to individuals.**




# ECONOMY ADULT SANI-MANIKIN


## Assemble Manikin

The parts needed to assemble a single manikin are: (1) manikin torso, (1) manikin head, (1) airway sample (2) pine tree push pins, (1) rubber bands, (2) ear pins, (2) washers, and (2) hairpin clips.


- Steps:**
1. Insert the pin into the ear and through the neck, and then add the washer and connecting pin on the inside of the manikin's head.
  2. Insert the pine tree push pins (2) into the back of the manikin.

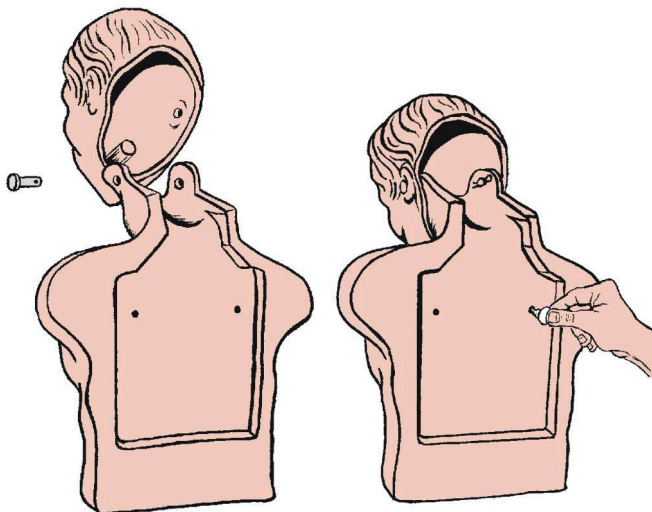


3. Place the rubber band over the pine tree push pins.
4. Fold the closed end of the airway lengthwise four times to simplify pushing the airway through the manikin's mouth. Tilt the head back and insert the closed end of the airway into and through the back of the tube inside the manikin's mouth.

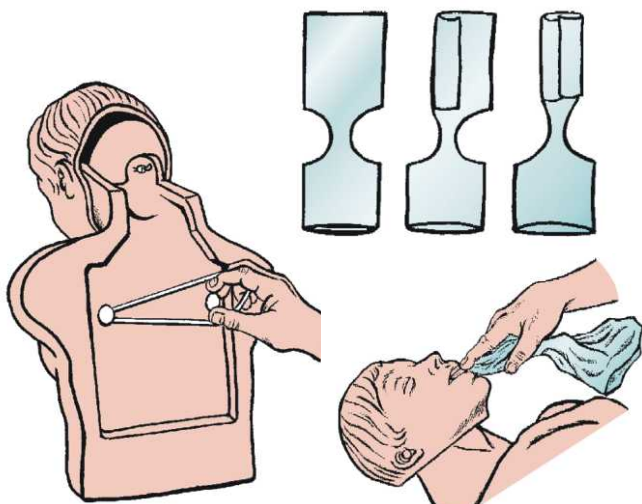

5. Pull the closed end of the airway down the back of the manikin, leaving approximately 2 inches remaining outside of the manikin's mouth to use as a face shield.
6. Flatten the closed end of the airway against the manikin's back and secure it under the rubber band.

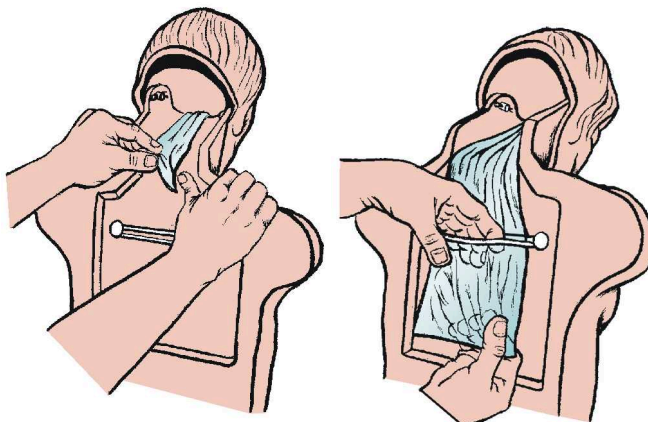

7. Spread the open end of the airway over the manikin's nose and chin. Finger sweep to open the bag to ensure there are no twists and that the airway is open. When the individual completes airway inflation have them remove the bag so the next student can install their airway. The airway can be retained by an individual for use during a training session, but it is necessary to discard the airway at the end of a session.

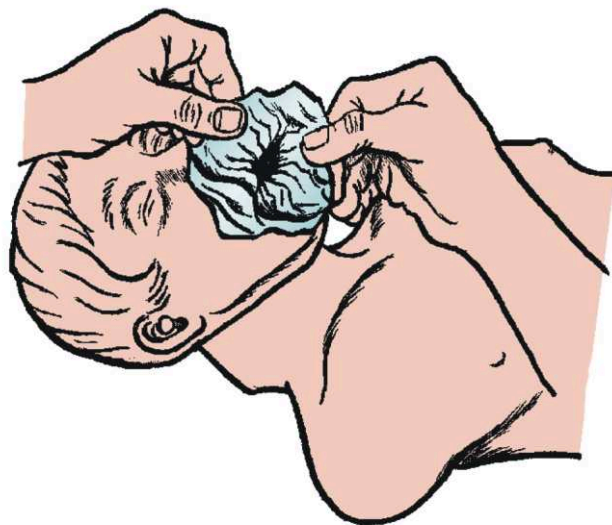

*(continued on next page)*



### Using the Economy Sani CPR Manikin

To ventilate the manikin use rescue breathing or a bag valve mask. An open airway and adequate ventilations will expand the airway lifting the manikin up off the floor. To eliminate cross contamination the airway is designed for a single user. Airways are to be disposed of at the end of the training session. Reduce adhesive build up from hands free pads by applying a thin coat of an automotive wax to the chest locations.

### Cleaning and Maintaining Economy Sani CPR Manikin

Remove used airway and dispose. Clean manikin with warm soapy water. Dry the manikin before storing. Do not immerse in fluids. Store in a dry location to avoid absorbing moisture.

### Replacement Parts

- 2135** Adult Sani Man Airway 100 Pack
- 2136** Replacement Head with Hardware
- PP1247** Pine Tree Push Pins
- 2145** Economy Sani CPR Manikin 4 Pack
- PP2800** Carry Bag Economy CPR Manikin 4 Pack
- 2153** Sani CPR Family 3 (Adults) 2 (Infants) Pack
- 2121** Sani CPR Baby
- 2124** Sani CPR Baby 4 Pack
- 2125** Sani Baby Airway 100 Pack
- 2140** Sani CPR Child Manikin
- 2141** Sani Child Airway 100 Pack

### Warranty

Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required.

**Do not allow print material to come in contact with the skin of your product. The ink will embed in the porous surface and be impossible to remove. Ink stains are not covered under warranty.**

### Return Policy

At no time will any product be accepted without proper return authorization issued by Simulaids. Please contact our Customer Service Department to arrange a return and obtain a RGA number. Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Serial number and invoice number from the agency through whom the product was purchased must be provided for warranty repairs. No return authorization number will be provided without this information.

Should you have any questions or wish further information on any product we manufacture call or write our Customer Service Department.



**Manufacturers of Training Manikins, Casualty Simulation Kits & Medical Training Devices**

PO Box 1289 • 16 Simulaids Drive • Saugerties, NY 12477  
(845) 679-2475 • 1-800-431-4310 • FAX: (845) 679-8996  
[www.simulaids.com](http://www.simulaids.com)